

# Radial Arm Board Drape

Compatible with the Adept STARBoard

Product Code: MRG100 | CS/50

## Warnings

This product is non-Sterile and meant for single-use only. Do not re-use.



1: Slide the drape over the STARBoard ensuring it is pulled all the way back.



2:

Place the patients arm on the Armrest, place the patient's hand on the wrist pivot handle. This will slide the drape into position.



3:

The drape includes two optional adhesive Strips to gently secure the arm or hand

### Product Support & Customer Service:

Tel: 309-263-6901

Email: [sales@mrg-ltd.com](mailto:sales@mrg-ltd.com)

Website: [www.mrg-ltd.com](http://www.mrg-ltd.com)



Nonsterile



Single-Use



Contains  
no Latex