

Approved Disinfectants

Approved Cleaners List

Approved Disinfectant by Brands

Liquids:

ORION Laboratories
70% Isopropyl Alcohol

Jaychem Industries
2% Chlorhexidine / 70% Alcohol

Betadine
Povidone Iodine (7.5%)

BODE Chemie GmbH
Kohrsolin FF

STERIS
Coverage Spray TB

CaviCide
Metrex Research

Dissolvable:

Du Pont
Rely+On Virkon Tablets

Surface Wipes:

PDI Sani-Cloth Bleach

PDI Sani-Cloth AF3

Reynard Health Supplies
Surface Disinfectant Wipes

Metrex Research
CaviWipes

Clinell Wipes



Disinfect with the approved cleaners below. Do not sterilize using Autoclave, EtO or Gamma.



Discoloration: Some disinfectants may cause slight discoloration to soft blue components. This will not affect the strength and will remain fit for purpose.

Approved Active Disinfectant Ingredients

Acid Based:

- ≤10% Malic Acid CAS 6915-15-7
- ≤6% Sulfamic acid CAS 5329-14-6

Alcohol Based:

- ≤5% 2-Butoxyethanol CAS 111-76-2
- ≤10% Butyldiglycol CAS 112-34-5
- ≤70% Isopropyl Alcohol (Propan-2-ol) CAS 67-63-0
- ≤10% Tridecanol CAS 69011-36-5
- <10% Alcohols, C12-14, ethoxylated CAS 68439-50-9

Sulphate Based:

- ≤0.1% PHMB CAS 27083-27-8
- ≤55% Potassium Peroxymonosulfate CAS 70693-62-8
- ≤3% Potassium Persulfate CAS 7727-21-1

Ammonium and Chloride Based:

- ≤5% Benzalkonium Chloride CAS 68424-85-1
- ≤0.28% Benzethonium Chloride CAS 121-54-0
- ≤10% Benzyl-C23-18-Alkyl-dimethyl Ammonium Chloride CAS 8001-54-5
- ≤2% Chlorhexidine CAS 55-56-1
- ≤10% Didecyl Dimethyl Ammonium chloride CAS 7173-51-5
- ≤0.5% Quaternary Ammonium Compounds CAS 68956-79-6
- <5% Benzyl-C 12-18 alkyldimethylammonium chlorides CAS 63891-01-5

Others:

- ≤10% Glutaral CAS 111-30-8
- ≤0.63% Sodium Hypochlorite CAS 7681-52-9
- ≤7.5% Povidone Iodine CAS 25655-41-8
- 0.5% Disodium Cocoampho Dipropionate CAS68604-71-7



Discoloration: Some disinfectants may cause slight discoloration to soft blue components. This will not affect the strength and will remain fit for purpose.