



Radial Arm Board Drape

Warnings

This product is non-Sterile and meant for single-use only. Do not re-use.



1.

Part 1:
Slide the drape over the STARBoard ensuring it is pulled all the way back.



2A.

2:
Place the patients arm on the Armrest, place the patient's hand on the wrist pivot handle. This will slide the drape into position.



2B.



3.

3:
The drape includes two optional adhesives Strips to gently secure the arm or hand

Product Support & Customer Service:

Tel: 309-263-6901

Email: sales@mrgona.com

Website: www.mrgona.com



Nonsterile



Single-Use



Contains no Latex