



## Instrument Tray Corner Protectors


### INSTRUCTIONS FOR USE:



Brand Name of Product	SteriProtect Tray Corner Protector
Product Code Number	MRG0400CP / AMO200PC
Purpose of Product	To protect the corner of sterilization wrap from sharp edges of sterilization trays
Range of Applications	Wrapped trays that are steam, ETO, and low temperature plasma sterilization
Specification of Product	Thermoplastic Elastomer (Polypropylene). Latex Free. BPA Free

Instructions For Use	
Description for Use	Place under corner of sterilization tray. Be sure all four corners are covered.
Diagram	

Reprocessing Instructions	
Manual Cleaning	Instrument Tray Corner Protectors may be manually cleaned with a neutral pH enzymatic detergent and rinsed with running water. You may use a soft bristle brush to remove any contaminants. Do not use scouring pads or abrasive cleaners.
Automated Washer	Instrument Tray Corner Protectors may be processed in an automated instrument washer.

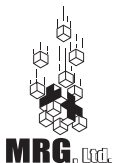

1920 South Main Street • Morton, IL 61550 • O: 309.263.6901 • T: 888.674.6621  
 F: 309.263.6903 • E: sales@mrgona.com • www.mrgona.com

**MEDICAL RESOURCES GROUP LIMITED**