

ADEPT MEDICAL APPROVED DISINFECTANTS

WARNING - DO NOT USE ANY CLEANING METHOD THAT EXCEEDS A TEMPERATURE OF 65 °C

APPROVED

LIQUIDS

ORION Laboratories - 70% Isopropyl Alcohol* - [MSDS](#)

*Do not use 70% Isopropyl Alcohol on STARBoard with an IBN product code lower than IBN001536

Jaychem Industries - 2% Chlorhexidine / 70% Alcohol (Pink) - [MSDS](#)

Betadine - Povidone Iodine (7.5%) - [MSDS](#)

C-Tec - Nature Clean (Diluted 1:10 with water)

BODE Chemie GmbH - Kohrsolin FF - [MSDS](#)

STERIS Coverage Spray TB - [MSDS](#)

CaviCide by Metrex Research - [MSDS](#)

SURFACE WIPES

PDI Sani-Cloth Bleach - [MSDS](#)

PDI Sani-Cloth AF3 - [MSDS](#)

- Reynard Health Supplies - Surface Disinfectant Wipes

- [MSDS](#)

Diversey - Oxivir Tb Wipes - [MSDS](#)

CaviWipes by Metrex Research - [MSDS](#)

Clinell Wipes - [MSDS](#)


DISSOLVABLE

Du Pont - Rely on Virkon Tablets - [MSDS](#)

NOT APPROVED

DIVERSEY VIREX TB - [MSDS](#)

This disinfectant cleaner is not approved by the manufacturer as it may cause damage to some of the STARSystem components after prolonged use.

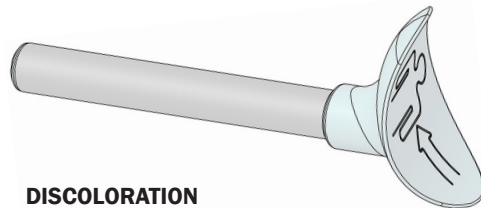

DISCOLORATION

Some disinfectants may cause slight discoloration to the soft blue parts on the STARSupport. They will not effect the strength and will remain fit for purpose.